


Apollo Pressure Activated Safety Lancet

Capillary Blood Sampling

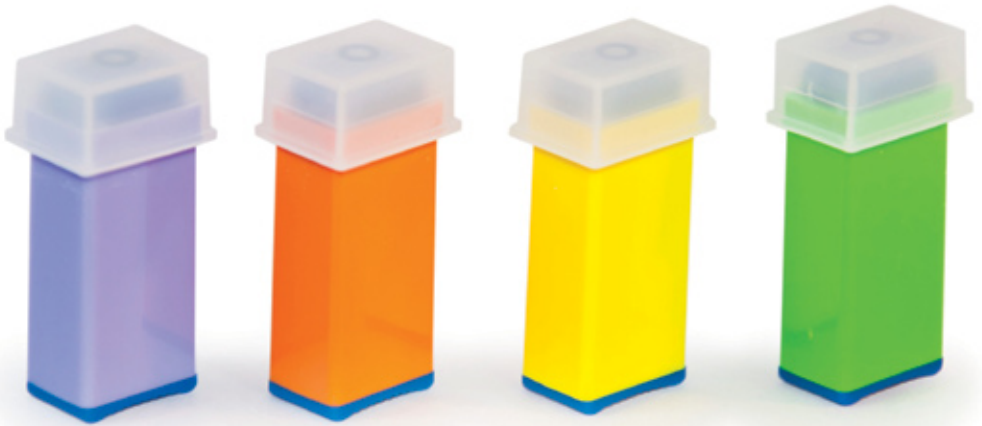

SAFE

Needle is fully shielded
before and after use.



SIMPLE

3 simple stages to operate:
1 Un-cap
2 Press
3 Dispose



**CLINICALLY
COST EFFECTIVE**

The most cost efficient
Lancet available on NHS
prescription and contract.

► Patient comfort

Patented, pressure activated linear needle track preventing needle vibrations and ensuring high speed penetration for maximum comfort.

► Totally sealed

Needle tip within medical rubber to maintain full sterilization.

► User safety

Is assured retractable fully shielded needle before and after use protects healthcare workers from needle-stick injuries.

► Single use

Lancet cannot be re-capped after use clear indication to the operator that it has been used and requires disposing in a sharps container.

► Accurate

Smooth tri-bevelled needle point, auto polished to ensure precision and reduction in penetration force required.

► Option




Colour coded choice of needle gauge sizes and penetration depths to meet most capillary blood requirements.

| Colour Code | Needle Gauge | Depth | Application | Product code | NHS prescribing PIP code |
|-------------|--------------|-------|--|--------------|--------------------------|
| Purple | 28g | 1.8mm | Low flow 10-20ul Glucose test | AMTL2818 | 366-0453 |
| Yellow | 26g | 1.8mm | Low flow 10-20ul Glucose test | AMTL2618 | 366-0461 |
| Light Green | 23g | 2.4mm | Med flow 20-40ul | AMTL2324 | 366-0479 |
| Orange | 21g | 2.4mm | Med flow 20-40ul | AMTL2124 | 366-0487 |
| Blue | 21g | 1.8mm | Med flow 20-40ul | AMTL2118 | TBA |
| Dark Green | 21g | 2.2mm | Med flow available on request | | |
| Plum | 21g | 2.8mm | High available on request | | |
| Light Blue | 18g | 1.8mm | High neonates Available on request | | |

HOW TO USE

Choose a suitable sampling site, then:


- 1** Carefully Pull off the protective cap.
Do not use if the protective cap has been previously removed.
- 2** Push the lancet firmly against the chosen site to release the lancet. If required, massage the sample site, taking care not to squeeze too hard at the site.
- 3** Single use needle retracts immediately after sampling, dispose in a suitable sharps container.

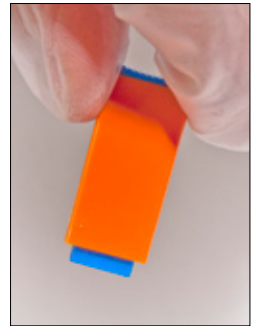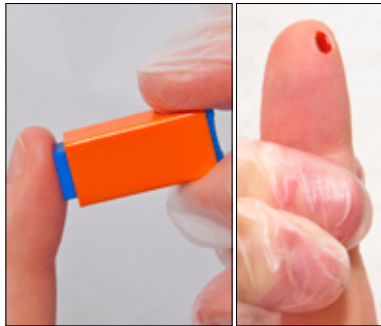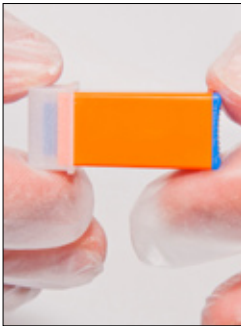

CLINICALLY COST EFFECTIVE

The most cost efficient Lancet available on NHS prescription and contract

(Cost comparison for a box of 100)

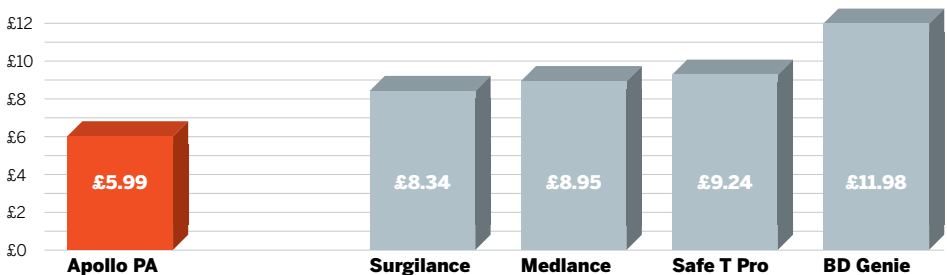

**RECEIVE
A SAMPLE
PACK**



Telephone
01636 831201
or Email:
apollomedicaltd1@btinternet.com

**FAX BACK TO
01636 831201**



Name _____

Clinic or practice **delivery** address _____

Postcode _____

Telephone _____

Clinic or practice **invoice** address _____

Postcode _____

I would like to order Apollo Pressure Activated Lancets as indicated above

Order No: _____

I would like to receive free Apollo Pressure Activated Lancet samples as shown above:

Paediatric Max Flow High Flow Normal Flow Low Flow Micro Flow

THE DETAILS GIVEN IN THIS LEAFLET ARE CORRECT AT THE TIME OF GOING TO PRESS.
THE COMPANY RESERVES THE RIGHT TO IMPROVE THE EQUIPMENT SHOWN.

Contact us for more information



Apollo Medical Technologies Ltd
Fox Briars, Marlock Close, Fiskerton,
Southwell, Nottinghamshire NG25 0UB

Telephone: **01636 831201**
Email: **supercheck2@btinternet.com**
Website: **www.supercheck2.co.uk**

Apollo Medical Technology
exclusive UK distributor of
Zhen Wu Medical products.

Manufactured by:
Suzhou Zhen Wu Medical.
248 Chijawan Road, Beishe
Community, Wujiang,
Suzhou, Jiangsu, P.R. China.

